



Simple User Guide for Patients

- 1) You have received the device fully charged, and as such can be used for a week without recharging. The device is recording only when turned on, and it turns off automatically within 2 minutes after the voiding is finished.
- 2) Insert the disc into plastic bag with a zip fastener and seal it.
- 3) Press the push-button that is located on the bottom side of the device (the side with the label) and a green LED lights up next to the push-button. If a red LED lights up instead, the device signals low battery charge – please bring the device back to the doctor, so that the device may be recharged. If a green LED lights up, place the device on a flat surface with the top side (the side without a label) facing up.
- 4) After pressing the push-button and placing the device on a flat surface, put a container on the top side of the device. The container must be placed on the device within 10 seconds after pressing push-button.
- 5) The device indicates start of the recording with an audio signal 10 seconds after pressing the push-button. Voiding into the container may begin. The device will record up to 60 seconds of voiding.
- 6) The device stops recording after 60 seconds, then waits 2 minutes for a connection with the evaluation unit. If there is no connection, the device turns off automatically. Recorded data stay in the memory of the device until downloaded successfully.
- 7) You can proceed with recording repeating the same steps: Start by pressing the push-button, place the device on a flat surface, and put a container onto the top side of the device. After the device beeps, the recording starts and the voiding may begin.
- 8) Unless stained, the protective plastic bag can be used for the whole monitoring period. In case of urine stains, disinfect the bag or replace it.
- 9) After the monitoring period is over, remove the device from plastic bag. Return the device to the doctor, and dispose of the protective bag after disinfecting it.

